SILVERBAND[®]

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www.silverbandgel.com

SILVERBAND[®] Wound Dressing Gel

PRODUCT DESCRIPTION:

Silverband[®] Wound Dressing Gel is a water-based gel wound dressing for use in moist wound care management. Silverband[®] Wound Dressing Gel contains silver that may help inhibit the growth of microorganisms within the dressing. Silverband[®] Wound Dressing Gel has been evaluated in standard tests that show it can reduce the levels of common microorganisms including *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Escherichia coli*, *Candida albicans* and *Aspergillus niger*. (USP <51>).

INDICATIONS FOR USE:

For the topical management of:

- 1st Degree Burns
- Minor Cuts
- Lacerations
- Abrasions
- Skin Irritations
- Minor Sunburns

DIRECTIONS FOR USE:

1. Wash your hands with soap and water before touching the wound.
2. Carefully clean the wound using soap and water or an appropriate commercial brand of wound cleanser.
3. Dispense Silverband[®] Wound Dressing Gel on to an appropriate clean applicator (such as a tongue depressor or gauze) in a sufficient amount to liberally cover the wound. Cover the gel with an appropriate secondary dressing cover such as a gauze bolster, dressing wrap, film or non-woven adhesive secondary dressing.

CHANGING THE DRESSING:

1. Carefully remove the secondary dressing from the wound.
2. Residual gel may be removed with the cover dressing.
3. Follow instructions above to re-apply the gel.

FREQUENCY OF CHANGE:

Silverband[®] Wound Dressing Gel should normally be changed every 24 to 72 hours, but the frequency will depend on the type and condition of the wound.

PRECAUTIONS:

1. Consult with a healthcare professional if you have any concerns or questions about the wound.
2. Be alert for signs of wound infection.

These can include:

Pus or cloudy fluid draining from the wound
Increased redness around the wound
Wound has become extremely tender
Increased pain or swelling 48 hours after the wound occurred
Onset of fever
Wound has not healed within 10 days from the date of the injury

3. Silverband[®] Wound Dressing Gel is designed for use on any part of the body except for oral and ophthalmic use. Consult a healthcare professional before using the gel in any other manner.

WARNINGS: Keep out of the reach of children.

Consult a Health Care Professional if you have any concerns or questions about the wound.

For External Use Only.

STORAGE: Store at room temperature.

MANUFACTURER: ABL Medical, LLC

705 East 50 South, American fork, UT 84003, USA

IMPORTER & LICENSE HOLDER: NSBT LTD.,

Me'ir Ya'ari 17, Tel Aviv 6937117

Customer service telephone number: 03-5441042

Approved by the Ministry of Health. Registration number in the Ministry of Health National Medical Device Registry: 27620002

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