INSTRUCTIONS FOR USE

This Instructions For Use leaflet contains important product use and safety information. Please read carefully and keep this for future use

PRODUCT NAME: SYSTANE™ COMPLETE Lubricant Eye Drops

DESCRIPTION: SYSTANE™ COMPLETE Lubricant Eye Drops is a sterile, preservative-free, white emulsion

containing propylene glycol (0.6%), hydroxypropyl guar, mineral oil, dimyristoyl phosphatidylglycrol, polycoyl 40 stearate, scrotitan tristearate, boricacid, sorbitol, and purified water. May contain hydrochloric acid and/or sodium hydroxide to adjust pH.

SYSTANE™ COMPLETE Lubricant Eye Drops is naturally white in appearance prior to applicati

CLINICAL BENEFITS:

en used as directed, SYSTANE™ COMPLETE Lubricant Eye Drops:

Provides eye surface protection.

Improves visual performance.

Helps to restore eye surface health. Provides dry eye symptom relief.

INTENDED PURPOSE/USE:

SYSTANE™ COMPLETE Lubricant Eye Drops is intended to lubricate the eye surface

INTENDED USERS/TARGET POPULATION:

Adults (18 years and dider) experiencing dry eye symptoms, who are capable of understanding and following the directions and instructions for use (or have a caregiver who can act on their behalf).

SYSTANE™ COMPLETE Lubricant Eye Drops is indicated for the temporary relief of burning and irritation in persons experiencing dry eye symptoms.

PERFORMANCE CHARACTERISTICS:

Based on the total body of evidence from clinical investigations, published clinical literature and market experience (post market surveillance), SYSTANETM COMPLETE Lubricant Eye Drops perform as indicated for the temporary relief of burning and irritation in persons experiencing dry eye symptoms.

DIRECTIONS:

Shake the bottle well before first opening.

Hold the bottle just below the cap and twist the cap to open.

BEFORE FIRST USE, hold the bottle at an approximate 90-degree angle and dispense one drop

of product to discard. Shake the bottle downward to remove residual product (see picture).

Shake well before using.

SYSTANE™ COMPLETE Lubricant Eye Drops may be used as needed throughout the day to relieve the symptoms of dry eye. Instill 1 or 2 drops in the affected eye(s) as needed and blink.

After use, before closing the cap on the bottle, shake the bottle down remove residual product that may be left on the tip (see picture).

CONTRAINDICATIONS (Reasons not to use):

People allergic to any ingredient in SYSTANE™ COMPLETE Lubricant Eye Drops should not use

WARNINGS:

It is essential that you follow your eye care professional's directions and all labeling instructions for proper use of the product. If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, stop using SYSTANE™ COMPLETE Lubricant Eye Drops and consult your eye care professional as the problem could become more serious.

Temporary blurred vision or other visual disturbances may affect the ability to drive or use machines. If blurred vision occurs after application, wait until the vision dears before driving or using machinery.

PRECAUTIONS:

For ocular use only.

TAMPER EVIDENT: Do not use if tamper evident cap is damaged or missing.

Do not use if package or bottle (sterile packaging) is damaged. To avoid contamination, never touch the dropper tip of the container to any surface.

If swallowed, get medical help.

If emulsion changes color, do not use

Replace cap after using. Keep container tightly closed when not in use.

This product contains a level of boric acid that is safe when used according to label

Use before the expiration date marked on the product.

Once the bottle is opened, discard any remaining emulsion as ins o For 3 mL fill size, discard remaining emulsion after 30 days.

o For 10 mL fill size, discard remaining emulsion after three months.

Store between 2°C to 30°C. 2°C 1 20°C

Keep out of the reach of children.

ADVERSE EFFECTS (Possible problems and what to do):
If you experience persistent eye discomfort, excessive tearing, vision changes or redness of the eye, stop using SYSTANE™ COMPLETE Lubricant Eye Drops and consult your eye care professional

Serious Incident Reporting:

Any serious incident related to the use of this medical device should be reported to Alcon Laboratories, Inc.: Contact the local country office or your Alcon distributor

Email: ga.complaints@alcon.com Website: https://www.alcon.com/contact-us

for users and/or patients in the European Economic Area and Switzerland, these serio ncidents should also be reported to the competent authority for medical devices of your State.

SYSTANE™ COMPLETE Lubricant Eye Drops is supplied in plastic bottles containing fill sizes of 3 mL or 10 mL. Not all pack sizes may be available

STERILE A Sterilized using aseptic processing techniques.

GLOSSARY TABLE FOR SYMBOLS

Symbol Definition: (Ii Consult instructions for use or consult electronic instructions for us (Do not use if package is damaged and consult instructions for us (A) Contains hazardous substances Single sterile barrier system MD Medical Device 0 Packaging waste license sign

6201 South Freeway Fort Worth, Texas 76134-2099, USA

EC REP Authorized Representative in the furgoean Union Lidsterveld 3 2870 Paurs-Sint-Amands

Lapidot Medical Import and Marketing Ltd. 8 Hachita Steet PO Rox 3552 Caesarea Industrial Park, 3088900 Customer service: 1700-70-90-33

© 2024 Alcon Inc.

Approved by the Ministry of Health, Medical Device Department. Registration No.: 4350687

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